**Elaboro GmbH**  
Hagenower Str. 73  
19061 Schwerin | Germany

TEL : +49 385 3993 300  
E-MAIL : info@elaboro.de

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## CERAFUSION® LiSi PURE | SHINE - Fusing spray glaze for zirconia

Raw ceramic for the manufacture of dental prostheses. Type, classification, and labeling according to DIN EN ISO 6872: Dental ceramics type I, class 1

### 1. Area of application

LiSi PURE is a ready-to-use, easy-to-apply lithium silicate glass ceramic for surface finishing of monolithic zirconia restorations. No pre-treatment of the surface is necessary - no bonding agents, solvents, or pre-firing are required.

The zirconia surfaces should be free of dust and grease to ensure an even coating.

LiSi PURE is suitable for all dental zirconia.

For high-quality individual work in combination with stains, we recommend the additional use of LiSi LOWFUSE.

### 2. General safety and work instructions

LiSi PURE is intended exclusively for use in dental laboratories by trained professionals. Please observe the instructions on the spray can:

- Flammable aerosol, keep out of reach of children
- Container is pressurized - may burst if heated
- Keep away from heat, hot surfaces, sparks, and open flames, no smoking
- Protect from sunlight; do not expose to temperatures exceeding 50°C / 122°F
- Do not pierce or burn. (even after use)
- Use only in well-ventilated areas, do not inhale spray mist
- Wear protective gloves, protective clothing, eye protection, and a mask, use workplace extraction if necessary
- Causes serious eye irritation; if eye irritation persists, seek medical advice
- Do not force open or burn after use
- Always empty spray cans completely
- Intraoral use is not permitted, refer to the safety data sheet

### 3. Storage and working conditions

- Store at room temperature
- Processing temperature: 15-35 °C / 59-95 °F
- Temperatures that are too high or too low will impair the spray result
- Use only in well-ventilated areas
- Use an extraction system and dust mask
- Ensure good lighting to detect uneven coating

### 4. Preparation of the zirconia restoration

The finished sintered and fitted zirconia restoration must be dry, clean, and free of dust and grease in order to achieve optimal results. A very thin layer of LiSi PURE is sufficient. For fully anatomical monolithic restorations, functional and aesthetic features such as chewing surfaces, contact points, color gradients, and effects should be taken into account before sintering the zirconia. Crown margins, marginal fits, and occlusion are practically unaffected by a single layer application. We recommend applying the final layer of LiSi PURE only after dental try-in and any necessary corrections.

### 5. Application

#### Preparation

- Shake the spray can vigorously for at least 1 minute before use until the mixing balls can be heard
- Attach the spray head with lance and shake the can again briefly
- Shake again after each spray pause to prevent nozzle clogging

#### Testing & spraying technique

- Before using for the first time, test spray (e.g., on glass)
- Spray distance: approx. 15-20 cm from the object
- **Apply in short bursts**
- Hold the spray can as upright as possible
- Only use the spray nozzle provided

### Tips for optimal use

- Wet spots or streaks indicate that the spray distance is too short
- Uneven powder deposits indicate overdosing or insufficient shaking
- Always apply LiSi PURE in short bursts
- The spray should only be applied to the extent that the zirconia surface still shimmers through the white powder

### 6. Troubleshooting before ceramic firing

Incorrectly sprayed parts can be easily washed off with water or steamed off. The same applies if too much material has been applied. After drying with compressed air, LiSi PURE can be sprayed on again. If small areas are damaged when handling the sprayed dental restorations, these can be easily sprayed over again with a thin layer.

### 7. Cleaning

Clean the spray head immediately after using the spray can (e.g., with lukewarm water and compressed air, or an ultrasonic bath if necessary). Then dry with compressed air.

### 8. Ceramic Firing

Perform the ceramic firing according to the specifications (firing parameters table). During the diffusion firing, components of the zirconium oxide and components of LiSi PURE diffuse into each other in the area close to the surface and form a strong bond during cooling. At the same time, a smooth surface is formed.

**a. Diffusion firing:** When used correctly, you will obtain a resilient, smooth, and homogeneous surface of the highest quality after just one firing (diffusion firing). At the same time, an optimal bond to the zirconium oxide is created.

After diffusion firing, the restoration can be sprayed again with LiSi PURE and further processed if necessary.

**b. Use of stains:** LiSi PURE can be used on its own or in combination with high-melting coloring agents that can withstand temperatures of up to 920 °C / 1688 °F. If the restoration is to be individually characterized, the stains are applied immediately after the first firing (diffusion firing).

*To reduce the firing temperature and preserve the color brilliance as much as possible, we recommend using LiSi LOWFUSE in this step.*

**c. Personalisation firing:** For older ceramic ovens, a simplified firing program without vacuum can be used as an alternative.

### 9. Troubleshooting after ceramic firing

The LiSi PURE coating is only 10-15µm thick when applied normally. If small pores are visible on the surface after firing, this is due to insufficient powder application. Spray the restoration again—without any surface treatment—and repeat the firing process. If necessary, check the oven calibration and firing parameters.

### Tips for the practitioner

During professional teeth cleaning (PTC), surfaces treated with LiSi PURE must not be blasted with powder materials. Roughening of the glass ceramic is irreversible and can increase susceptibility to plaque.

Please also note the information on workplace exposure, transport, and storage in the safety data sheet.

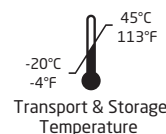
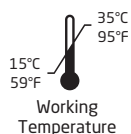
### CONDITION FIRING FOR ADHESIVE SURFACES

Closing Time	Standby Temperature		Heating Rate		Firing Temperature		Holding Time	Heating Rate		Firing Temperature		Holding Time	Long-term Cooling	Cooling Rate		Vacuum	
	min	°C	°F	°C/min	°F/min	°C		°F	min	°C/min	°F/min			°C	°F	min	depends on size
1	400	752	40	72	820	1508	5	20	36	920	1688	5	yes	20 - 80	35 - 140	max. 30%	

### ALTERNATIVE FIRING RECOMMENDATIONS FOR OLDER CERAMIC OVENS

Closing Time	Standby Temperature		Heating Rate		Firing Temperature		Holding Time	Long-term Cooling	Cooling Rate		Vacuum
	min	°C	°F	°C/min	°F/min	°C			°F	min	
1	400	752	40	72	920	1688	5	yes	20 - 80	35 - 140	no

### Explanation of danger and warning signs

Caution! Flammable Aerosol Causes severe eye irritation!

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©2015-2026, Elaboro GmbH